
**ENGLISH : Single-use filtering face-piece respirators**
**DEUTSCH : Filterierende Halbmasken**
**FRANÇAIS : Masques respiratoires a usage unique**
**NEDERLANDS : Filterrende halfmaskers**
**ITALIANO : Respiratore facciale filtrante monouso**
**ESPAÑOL : Mascarillas autofiltrantes desechables**
**PORTUGUÊS : Respiradores filtrantes**
**SVENSKA : Filterande andningsskydd**
**DANSK : Filtrende ansigtsmaske**
**NORSK : Filtrende masker**
**SUOMEKSI : Kertakäyttöinen suodattava hengityssuojain**
**MAGYAR : Egyszer használatos, szűrős légző-arcmaszk**
**ČECH : Filtrové lícní respirátory na jedno použití**
**POLSKI : Maski ochronne jednorazowego użytku**
**SLOVENE : Respirator, Za enkratno uporabo**
**BULGARE: Респиратор, За еднократна употреба**
**HRVATSKI: Jedna upotreba respiratornih maski**
**ESTONIEN : Respiraator, Ühekordsed**
**LATVIEŠU: vienreizējās lietošanas elpošanas maskas**
**LITUANIEN : Respiratorius, Vienkartiniai**
**ROUMAIN : Mască de gaze, Disponibil**
**РУССКИЙ: ИНСТРУКЦИЯ ПО ЭКСПЛУАТАЦИИ**
**ΕΛΛΗΝΙΚΑ: Αναπνευστικές μάσκες μίας χρήσης**
**لبليل الاستعمال : العربية**
**ЕНГЛЕСКИ ЈЕЗИК : УПУТСТВО ЗА УПОТРЕБУ**
**SRPSKI : Respirator, Za jednokratnu upotrebu**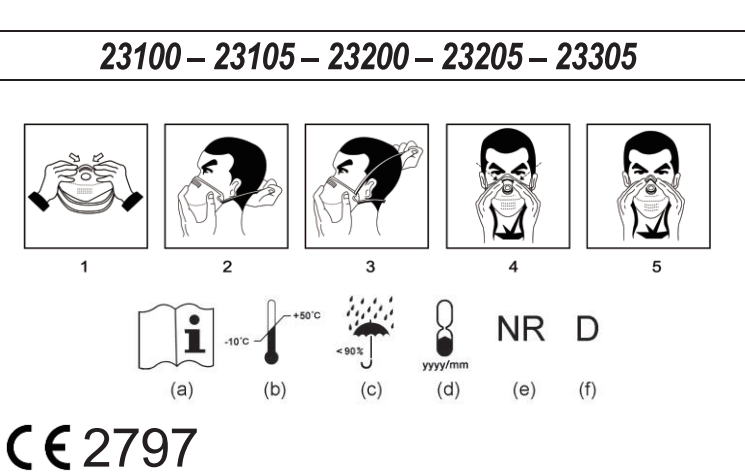

**BSI Group The Netherlands B.V.**  
**Say Building, John M. Keynesplein 9, 1066 EP**  
**Amsterdam, Netherlands**

**\*\*\* [https://wep.ovh/files/declaration\\_conformity/](https://wep.ovh/files/declaration_conformity/)**

**ENGLISH : ***INSTRUCTIONS FOR USE*****

**IMPORTANT :** PPE Regulation (EU) 2016/425, EN 149:2001+A1:2009 single-use respirators against solid and liquid aerosols are classified according to one of three categories - FFP1, FFP2, and FFP3. Before reading the following information, please check the category to which the respirator belongs - this is indicated on the packaging and on the respirator.

**APPLICATIONS : • FFP1 :** protection against non-toxic solid and liquid aerosols (e.g. oil-mists) in concentrations up to 4 x MAC/OEL/TLV, [i.e. NPF=4] or 4 x WEL, [i.e. APF=4].
**• FFP2 :** protection against non-toxic and low-to-average toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 12 x MAC/OEL/TLV, [i.e. NPF=12] or 10 x WEL, [i.e. APF=10].
**• FFP3 :** protection against non-toxic, low-to-average toxicity and high toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 50 x MAC/OEL/TLV, [i.e. NPF=50] or 20 x WEL, [i.e. APF=20].
**DO NOT USE** this respirator against vapour/gas in concentrations more than 1 x MAC/OEL/TLV, [i.e. NPF=1] or 1 x WEL, [i.e. APF=1].
**\*NOTE :** MAC = Maximum Admissible Concentration. OEL = Occupational Exposure Limit. TLV = Threshold Limit Value. NPF = Nominal Protection Factor. WEL = Workplace Exposure Limit. APF = Assigned Protection Factor.

**CHECK PRIOR TO USE :** Inspect respirator, including any component parts, for holes, tears or damage prior to use. Should any damage be identified, ***DO NOT USE IT***. Get a new one.

**FITTING INSTRUCTIONS : 1.** Place fingers inside the respirator. Bend the nose clip around fingers to form nose shape.
**2.** Holding mask in position over nose and mouth, with retaining straps around back of hand. Pull lower portion of strap over head and fit around neck.
**3.** Stretch and pull the upper strap over your head on the back of your head above your ears.
**4.** Using both hands, form nose clip around nose and adjust the tension in the upper and lower portions of the continuous loop head strap, to ensure a positive face seal.
**5.** (a) To test fit for respirator without exhalation valve - Cup both hands over the respirator and exhale vigorously.
**(b)** To test fit for respirator with exhalation valve - Cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the headband for better fit.
**\*NOTE :** Change respirators immediately if breathing becomes difficult or respirator become damaged or distorted, or a proper face fit can not be maintained. Respiratory protection is only effective if it is correctly selected, fitted, and worn throughout the time when the wearer is exposed to hazards.

**WARNING : 1.** Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death.
**2.** A properly selected respirator is essential to protect your health. Before using this respirator, consult an Industrial Hygienist or Occupational Safety Professional to determine the suitability for your intended use.
**3.** This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19%.
**4.** Do not use when concentrations of contaminants are immediately dangerous to health or life.
**5.** Do not use in explosive atmospheres.
**6.** Leave work area immediately and return to fresh air if (a) breathing becomes difficult, or (b) dizziness

or other distress occurs.
**7.** Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this respirator.
**8.** Never alter or modify this respirator in any way.
**9.** Only for single shift use. No maintenance necessary. Discard the used respirator after single shift use.
**10.** Keep respirators in the display box away from direct sunlight until use.

**Manufactured/Imported by :** WEP – 555 route de la dombes. 01700 Les Echets. Miribel. France.
The declaration of conformity is available on the web site see: \*\*
**MEANING OF PICTOGRAMS/SYMBOLS :** (a) See user information. (b) Temperature range of storage condition. (c) Maximum relative humidity of storage condition. (d) End of shelf life. yyyy = Year, mm = Month. (e) NR : this respirator is limited to single shift use only (with a maximum of 8 hours) and shall not be re-used. (f) D : this respirator passes the optional Dolomite clogging test for more comfortable breathing.

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